



# National Standard EZ-Cleans Plus® Kit Directions

---

**Kit Numbers: (25000, 25005)**

**Kit Contents:**

- Pair of Vinyl Gloves
- Odor Mask
- Red Z® Solidifier (10g. Pouch)
- Scoop/Scraper
- SaniZide Plus® Germicidal Wipe (Dimensions: 8" x 8")
- Red Bio-hazard Waste Bag (Dimensions: 24" x 24")
- White Disposal Bag (Dimensions: 30" x 36")
- Twist Ties (Quantity: 2)
- p.a.w.s.® Antimicrobial Hand Wipes (Quantity: 2 / Dimensions: 5" x 8")
- and Instructions

**Directions:**

- 1.) Put on odor mask and gloves.
- 2.) Sprinkle Red Z® over spilled area. Allow liquid to congeal for safer handling and transport
- 3.) Remove gelled material with scoop and scraper. Carefully place in the proper white disposal and/or red bio-hazard bag provided, dependent upon your local, state and federal Regulations for Infected solid waste.
- 4.) Use SaniZide Plus® Disinfectant wipe to clean contaminated surface.
- 5.) After procedure is completed remove gloves and mask. Place all contaminated materials in the proper bag. Seal bag with supplied twist ties. Dispose of bag as required by local, state and federal regulations for Infected solid waste.
- 6.) Wipe hands with p.a.w.s.® antimicrobial hand wipe. Wash with soap and water as soon as possible.