

Instruction Guide

EZ-Cleans® Plus Kit

Reorder # 17120 (Plastic Bag) | 17121 (Poly Bag) *17120 Does not include SaniZide Pro 1® Surface Disinfectant Wipe.*




What's Included:

- 10 g Red Z® Solidifier
- Instructions
- Pair of Vinyl Gloves
- Paper Towel
- p.a.w.s.® Antimicrobial Hand Wipe
- Red Biohazard Bag (24" x 24")
- SaniZide Pro 1® Surface Disinfectant Wipe
- Scoop/Scraper
- Twist Tie



Scan here to view the instruction video!

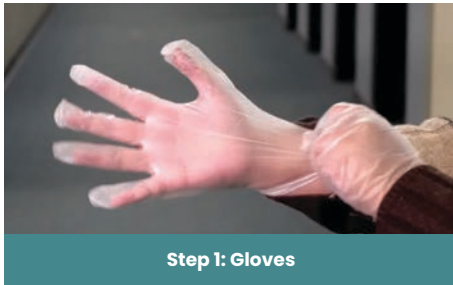

Step 1: Gloves

Step 1: Put on Vinyl Gloves.



Step 2: Red Z® Solidifier Pouch

Step 2: Sprinkle 10 g Red Z® Solidifier over the spill. Allow liquid to congeal for safer handling and transport.




Step 3: Scoop & Scraper

Step 3: Collect and remove gelled material with Scoop/Scraper.



Step 4: Red Biohazard Bag

Step 4: Place gelled material along with Scoop/Scraper into Red Biohazard Bag.

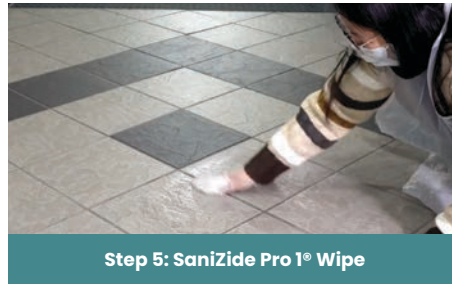


Step 5: SaniZide Pro 1® Wipe

Step 5: Clean the contaminated surface with SaniZide Pro 1® Surface Disinfectant Wipe.



Step 6: Red Biohazard Bag

Step 6: Place SaniZide Pro 1® Surface Disinfectant Wipe and Vinyl Gloves into Red Biohazard Bag.

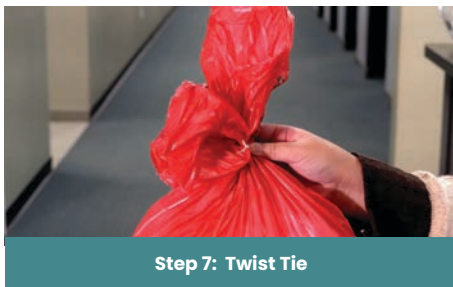

Step 7: Twist Tie

Step 7: Use Twist Tie to securely close Red Biohazard Bag. Dispose according to local, state, and federal regulations.



Step 8: p.a.w.s.® Antimicrobial Hand Wipe

Step 8: Use p.a.w.s.® Antimicrobial Hand Wipe to clean hands, then wash with soap and water as soon as possible.

Rev 25.10