

Instruction Guide

EZ-Personal Protection Kit

Reorder # 17606 (Plastic Bag)



What's Included:

- Combo Mask/Safety Shield
- Instructions
- Pair of Vinyl Gloves
- p.a.w.s.® Antimicrobial Hand Wipe
- Protective Gown
- Red Biohazard Bag (24" x 24")
- Twist Tie



Scan here to view the instruction video!




Step 1: Gown

Step 1: Put on Protective Gown.



Step 2: Face Shield

Step 2: Put on Full Face Safety Shield.

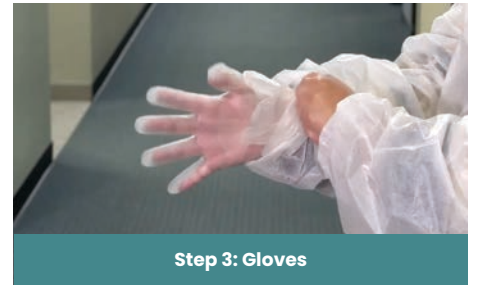


Step 3: Gloves

Step 3: Put on Vinyl Gloves.



Step 4: Red Biohazard Bag

Step 4: Place hazardous materials along with the Protective Gown, Full Face Safety Shield, and Vinyl Gloves into the Red Biohazard Bag.

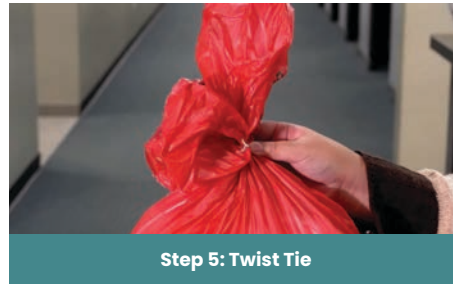

Step 5: Twist Tie

Step 5: Use Twist Tie to securely close Red Biohazard Bag. Dispose according to local, state, and federal regulations.



Step 6: p.a.w.s.® Antimicrobial Hand Wipe

Step 6: Wipe hands thoroughly with p.a.w.s.® Antimicrobial Hand Wipe then wash with soap and water as soon as possible.

Rev 25.10