

EZ-Personal Protection Kit Directions

Kit Number: (17606)

Kit Contents:

- Pair of Vinyl Gloves
- Fluid-Resistant Gown
- Combo Mask/Safety Shield
- Red Bio-hazard Waste Bag (Dimensions: 24" x 24")
- Twist Tie
- p.a.w.s.[®] Antimicrobial Hand Wipes (Quantity: 1 / Dimensions: 5" x 8")
- Instructions

Directions:

1. Put on fluid-resistant gown, combination safety mask and shield and gloves.
2. After procedure is completed remove gloves, safety mask/shield and gown... Place all contaminated materials in the red bio-hazard bag. Seal bag with supplied twist tie.
3. Dispose of bag as required by local, state and federal regulations for infected solid waste.
4. Wipe hands with p.a.w.s.[®] Antimicrobial Hand Wipe. Wash with soap and water as soon as possible.