

Instruction Guide

Hazardous Drugs Spill Kit

Reorder # 49725 (Plastic Bag)



What's Included:

- 21 g Green-Z® Fluid Solidifier
- Caution Sign
- Full-Face Safety Shield
- Identification Tag
- Instruction Card
- Scoop/Scraper
- Shoe Covers, Non-Slip
- Twist Tie
- USP <800> Compliant Gown
- 2 Pairs of USP <800> Compliant Nitrile Gloves
- 2 Wiper Pads (11" x 11")
- Yellow Waste Bag (20" x 13" x 39")



Scan here to view the instruction video!



Step 1: Shoe Covers

Step 1: Put on Non-Slip Shoe Covers.



Step 2: Gown

Step 2: Put on <USP> 800 Compliant Gown.




Step 3: Gloves

Step 3: Put on USP <800> Compliant Nitrile Gloves.



Step 4: Face Shield

Step 4: Put on Full-Face Safety Shield.

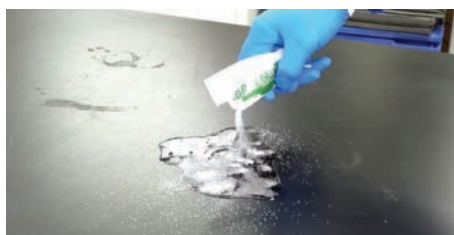

Step 5: Green-Z® Solidifier Pouch

Step 5: Sprinkle 21 g Green-Z® Solidifier over the spill. Allow liquid to congeal for safer handling and transport.



Step 6: Scoop & Scraper

Step 6: Collect and remove gelled material with Scoop/Scraper.



Step 7: Wiper Pads

Step 7: Wipe up any residue left behind with Wiper Pads.



Step 8: Yellow Waste Bag

Step 8: Place hazardous materials along with Shoe Covers, Gown, Gloves, and Face Shield into Yellow Waste Bag.



Step 9: Identification Tag & Twist Tie

Step 9: Use Twist Tie to securely close Yellow Waste Bag and attach Identification Tag. Dispose according to local, state, and federal regulations.

Rev 25.10