



Ready to Use



Odor Eliminating



No Bleach

# SaniZide Pro 1® Wipes

## 1-Minute Surface Disinfectant



Safetec's SaniZide Pro 1® is a broad-spectrum, EPA-registered surface disinfectant that combines cleaning and disinfecting in one simple step with its ready-to-use, alcohol-based, hospital-grade formula. Disinfecting is easier and safer with true one-minute kill claims—effective against 48 microorganisms and designed for repeated use on hard, non-porous surfaces. Great for use in medical settings, animal care facilities, salons, and at home.

SaniZide Pro 1® wipes align with the American National Standard (ANSI Z308.1-2021 – Section B2: Supplemental First Aid Supplies for Consideration), which outlines additional items that may be appropriate depending on the specific needs of a workplace.

Our disinfectant wipes deliver the same proven protection as our spray in a convenient, individually wrapped wipe. The bulk option is ideal for kit packers or for having it on hand for quick disinfecting in an emergency.

### Ideal for use in:

- Ambulances
- Ambulatory Surgical Centers
- Animal Care Facilities
- Bathrooms
- High Touch Surfaces
- Hospitals
- Laboratories
- Medical or Dental Offices
- Nursing Homes
- Office Buildings
- Other Healthcare Related Institutions
- Public Spaces
- Salons

Order No.	Size	Product Type	Carton/Canister	Case Qty.	Case Wt.
35923	6" x 6.75"	Wipe Canister	160 ct.	12/case	22 lbs.
35926	8" x 8"	Individual Wipe	50 ct.	6/case	8 lbs.
35927	8" x 8"	Individual Wipe	Bulk	500/case	15 lbs.

ANSI Supplemental Product

# 1-STEP 1-MINUTE

Effective Against MRSA, VRE, Tuberculosis, Influenza, Poliovirus, Norovirus, Monkeypox, SARS-CoV-2, & Hantavirus



EPA Reg. No. 88494-3-67161



Safetec of America Inc  
1-800-456-7077 | safetec.com

# SaniZide Pro 1® Wipe Efficacy Sheet

## May be used on hard, non-porous surfaces in:

**Hospitals and healthcare settings, including:** Ambulances, Anesthesia, CAT Labs, CCU, Central Supply, Clinics, Dental Offices, Dialysis Clinics, Doctor's Offices, Emergency Rooms, Home Health Care, Hospices, Laboratory, Laundry Rooms, Long Term Care Facilities, Newborn Nursery, Nursing Homes, Offices, Operating Rooms, Pediatrics, Physical Therapy, Physician's Offices, Ophthalmic Offices, Radiology, Recovery Rooms, Rehabilitation, Respiratory Therapy, Ultrasound, Transport Vehicles, Waiting Areas, X-Ray. **Critical Care Areas:** Cardiac Care Units, Emergency Rooms, Intensive Care Units, Neonatal Intensive Care Units, Pediatric Intensive Care Units, Surgery, Surgical Prep, and Surgical Recovery Units. **Animal Housing/Holding Areas:** Animal Hospitals, Animal Life Science Laboratories, Animal Transportation Vehicles, Breeding Establishments, Grooming Establishments, Kennels, Pet Shops, Tack Shops, Veterinary Clinics, Veterinary Examination Rooms, Veterinary Operating Rooms, Veterinary Waiting Rooms, Veterinary X-Ray Rooms, Zoos.

## Test Organism

## Contact Time

<i>Acinetobacter baumannii</i> MDR .....	1 minute
††Adenovirus .....	1 minute
†Avian Influenza.....	1 minute
<i>Bordetella bronchiseptica</i> .....	1 minute
<i>Bordetella pertussis</i> .....	1 minute
<i>Burkholderia cepacia</i> .....	1 minute
<i>Campylobacter jejuni</i> .....	1 minute
<i>Candida albicans</i> .....	1 minute
†Canine distemper virus .....	1 minute
††Canine parvovirus .....	1 minute
<b>Carbapenem resistant Enterobacter cloacae (CRE) or NDM-1 Positive .....</b>	<b>1 minute</b>
†Cytomegalovirus .....	1 minute
<i>Enterobacter aerogenes</i> or <i>Enterobacter cloacae</i> .....	1 minute
††Enterovirus EV-D68.....	1 minute
<i>Escherichia coli</i> - ESBL Producing MDR or NDM-1 Positive .....	1 minute
<i>Escherichia coli</i> (E. coli) or E. coli O157:H7 .....	1 minute
†Feline leukemia virus .....	1 minute
†Hepatitis B Virus (HBV) .....	1 minute
†Hepatitis C Virus (HCV) .....	1 minute
†Herpes Simplex Virus.....	1 minute
†HIV-Aids .....	1 minute
<b>†Influenza A and/or Pandemic 2009 H1N1 Influenza A, or H3N3 or Hong Kong Flu, or H2N2 .....</b>	<b>1 minute</b>
<b>†Influenza B .....</b>	<b>1 minute</b>
<i>Klebsiella pneumoniae</i> .....	1 minute
<i>Klebsiella pneumoniae</i> -ESBL Producing MDR or KPC-2 or CRKP or NDM-1 Positive .....	1 minute
<i>Legionella pneumophila</i> .....	1 minute
<i>Listeria monocytogenes</i> .....	1 minute
<b>†Monkeypox.....</b>	<b>1 minute</b>
<b>Mycobacterium bovis (Tuberculosis).....</b>	<b>1 minute</b>
††Norovirus .....	1 minute
††Poliovirus.....	1 minute
<i>Proteus vulgaris</i> .....	1 minute
<i>Pseudomonas aeruginosa</i> .....	1 minute
†Respiratory Syncytial virus (RSV) .....	1 minute
††Rhinovirus .....	1 minute
††Rotavirus.....	1 minute
<i>Salmonella enterica</i> .....	1 minute
<b>†SARS-CoV-2 (Coronavirus) .....</b>	<b>1 minute</b>
<i>Shigella dysenteriae</i> .....	1 minute
<i>Staphylococcus aureus</i> .....	1 minute
<b>Staphylococcus aureus - Methicillin resistant MRSA (Healthcare of Community Acquired) or MRSE .....</b>	<b>1 minute</b>
<i>Staphylococcus aureus</i> - Vancomycin intermediate VISA .....	1 minute
<i>Staphylococcus aureus</i> - Vancomycin resistant VRSA.....	1 minute
<i>Streptococcus pneumoniae</i> .....	1 minute
<i>Streptococcus pyogenes</i> .....	1 minute
<i>Trichophyton mentagrophytes</i> (Ringworm, Athlete's Foot, Jock Itch) .....	1 minute
†Vaccinia virus .....	1 minute
<b>Vancomycin resistant Enterococcus faecium or faecalis (VRE) .....</b>	<b>1 minute</b>
†Enveloped/lipophilic type virus	
††Non-enveloped/hydrophilic type virus	

Safetec's SaniZide Pro 1® products have demonstrated effectiveness against emerging viral pathogens similar to the Monkeypox virus and Hantavirus on hard, nonporous surfaces when used in accordance with the directions according to the label. A list of approved products can be found on the EPA's Disinfectants for Emerging Viral Pathogens (EVPs): List Q.

Safetec's SaniZide Pro 1® Products have demonstrated effectiveness against SARS-CoV-2 virus on hard, nonporous surfaces when used in accordance with the directions according to the label. A list of approved products can be found on the EPA's Disinfectants for Emerging Viral Pathogens (EVPs): List N.

SaniZide Pro 1® contains Ethanol (CAS 64-17-5) as a disinfectant, which in California is considered to be carcinogenic to humans when ingested in the form of an alcoholic beverage [IARC 1]. SaniZide Pro 1® also contains the following ingredients which do not appear on any listing of chemicals of concern as designated by the State of California or the European Union: Purified Water (CAS 7732-18-5) as a diluent, Propylene Glycol (CAS 57-55-6) as a solvent, Citric Acid (CAS 77-92-9) as a pH adjuster, Decyl Glucoside (CAS 68515-73-1) as a surfactant, Didecylidimonium Chloride (CAS 7173-51-5) as a disinfectant, Lauryl Glucoside (CAS 110615-47-9) as a surfactant.