

Instruction Guide

Universal Precaution Compliance Kit

Reorder # 17100 (Poly Bag) | 17102 (Hard Case)



What's Included:

- 10 g Red Z[®] Solidifier
- Combo Mask/Safety Shield
- Identification Tag
- Instructions
- Pair of Vinyl Gloves
- p.a.w.s.[®] Antimicrobial Hand Wipe
- Protective Apron
- Red Biohazard Bag (24" x 24")
- SaniZide Pro 1[®] Wipe
- Scoop/Scraper
- Twist Tie



Scan here to view the instruction video!




Step 1: Gown

Step 1: Put on Protective Gown.



Step 2: Face Shield

Step 2: Put on Full Face Safety Shield.

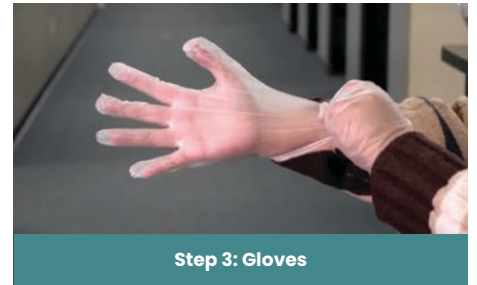

Step 3: Gloves

Step 3: Put on Vinyl Gloves.




Step 4: Solidifier

Step 4: Sprinkle 10 g Red Z[®] Solidifier over the spill. Allow liquid to congeal for safer handling and transport.



Step 5: Scoop & Scraper

Step 5: Collect and remove gelled material with Scoop/Scraper.

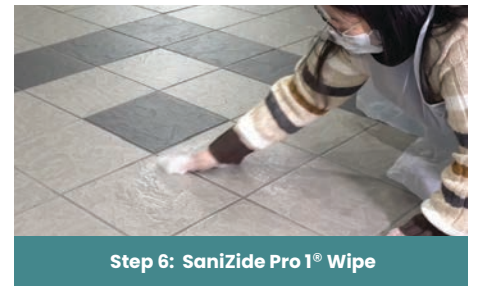


Step 6: SaniZide Pro 1[®] Wipe

Step 6: Clean the contaminated surface with SaniZide Pro 1[®] Surface Disinfectant Wipe.



Step 7: Red Biohazard Bag

Step 7: Place gelled material, Protective Gown, Full Face Shield, Vinyl Gloves, Scoop/Scraper, & SaniZide Pro 1[®] Surface Disinfectant Wipe into Red Biohazard Bag.

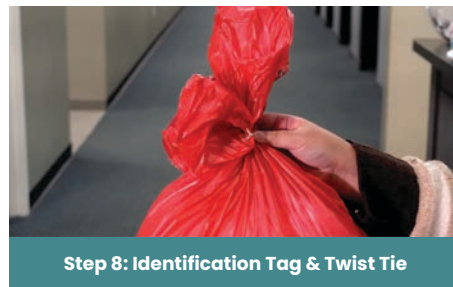

Step 8: Identification Tag & Twist Tie

Step 8: Use Twist Tie to securely close Red Biohazard Bag and attach Identification Tag. Dispose according to local, state, and federal regulations.



Step 9: p.a.w.s.[®] Antimicrobial Hand Wipe

Step 9: Wipe hands thoroughly with p.a.w.s.[®] Antimicrobial Hand Wipe then wash with soap and water as soon as possible.

Rev 25.10